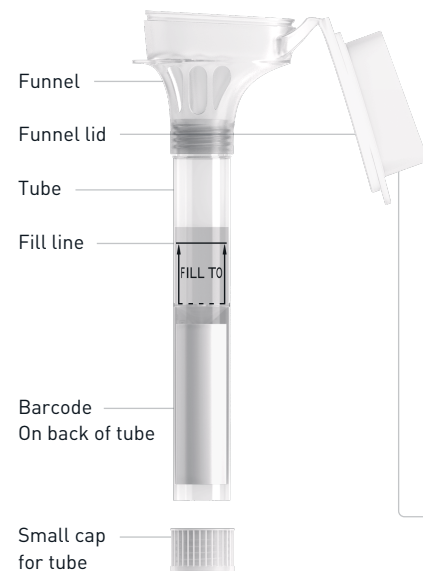
Storage 15°C $\nearrow$ 30°C
Shelf life 30 months *

* See use by date on collection device label

Kit contents

Contains 1 DNA collection kit. Volume 2 ml.

Intended use: for the collection of human DNA from saliva samples.

Contents: kit contains stabilising liquid.

Summary and explanation of the kit: Oragene®•DNA is a self-collection kit that provides the materials and instructions for collecting and stabilising saliva specimens.

Warnings & precautions

Do not eat, drink, smoke or chew gum for 30 minutes before giving your saliva sample

Do not remove the plastic film from the funnel lid

Do not close the funnel lid until you have provided enough saliva to reach the Fill line

Do not drink the stabilising liquid which is inside the funnel lid

- Wash with water if stabilising liquid comes in contact with eyes or skin. Do not ingest. See MSDS at www.atlasbiomed.com/safety for full information
- Small cap may pose a choking hazard

Atlas products are for scientific and educational purposes only. They are not a diagnostic tool. Consult a healthcare professional with any medical questions concerning the results of your Atlas DNA Test. The Atlas DNA Test is not indicated for paediatric, prenatal or preimplantation purposes.

Label legend

	Consult package insert
	Collect saliva by (Use by)
	In vitro diagnostic medical device
	Catalog number
	CE marking
	Caution, consult instructions for use
	Storage instructions
	Authorised representative
	Manufacturer
	Lot number

Emergo Europe
 (DNA Genotek Inc.)
 Prinsessegracht 20, 2514 AP
 The Hague, The Netherlands

Atlas Biomed Group Ltd.,
 Tower Bridge House, St Katharine's Way,
 London E1W 1DD, United Kingdom

Our team is here to help. If you have any questions, contact us at hello@atlasbiomed.com or give us a phone call: +44(0)20 7228 9286
 Fax: +44(0)20 7228 9286
www.atlasbiomed.com

For In Vitro Diagnostic Use
 OG-500.042

Made in Canada for Atlas
 by DNA Genotek Inc.
 3000-500 Palladium Drive,
 Ottawa, ON, Canada K2V 1C2
 Subsidiary of OraSure Technologies, Inc.
 Oragene® is a registered trademark of DNA Genotek Inc.
 Patent (www.dnagenotek.com/legalnotices)
 ©2017 DNA Genotek Inc., a subsidiary of
 OraSure Technologies, Inc., all rights reserved.
 PD-PR-00681 Issue1/2017-05

Instructions for DNA Test

Please note that it usually takes 2 to 5 minutes to provide a sufficient saliva sample

This product is only for people who are aged 18 and over

Multilingual instructions:
atlasbiomed.com/DNA-instructions

1

Register online

Register your kit at www.atlasbiomed.com/register using the unique barcode on the side of the collection tube.

Internet connection and active email address required.

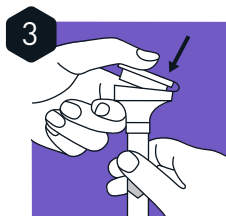
2

Collect

Read all instructions prior to collection.

Do not eat, drink, smoke or chew gum for 30 minutes before giving your saliva sample.

Spit into the funnel until the amount of liquid saliva (not bubbles) reaches the fill line.


3

Close the funnel lid

Hold the tube upright with one hand. Close the funnel lid with the other hand (as shown) by firmly pushing the lid until you hear a loud click. The liquid in the lid will be released into the tube to mix with the saliva.

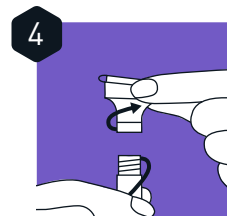
Make sure the lid is closed tightly.

4

Remove funnel

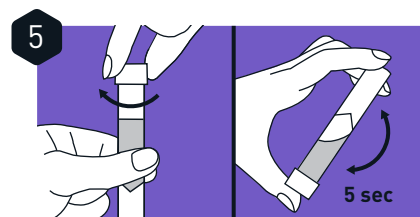
Hold the tube upright. Unscrew the funnel from the tube.

5

Cap tube & shake well

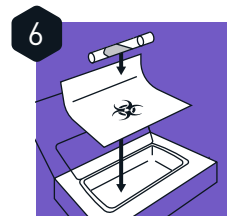Use the small cap to close the tube tightly. Shake the capped tube vigorously for 5 seconds to combine the saliva with the stabilising liquid. Discard funnel.

6

Seal in the bag and then place in the box

Place the collected sample into the bio-specimen bag and seal firmly. Put the bio-specimen bag containing the sample into the original box.

Do not remove the absorbent pad inside.

7

Seal the box

Close the box lid and seal shut.

8

Send

Mail sample for processing. Postage is prepaid.

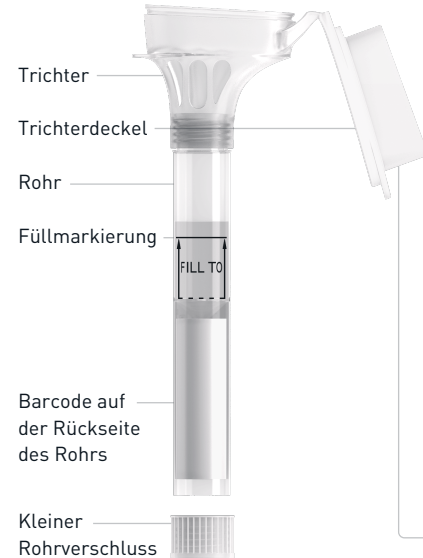
9

Check your results

Your results will be added to your personal profile on the website as soon as they have been processed.

This usually takes ☹ 4–5 weeks. You will be informed by email when the results are online. You can log in to www.atlasbiomed.com/profile to access your results.

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© 2018 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.